


## Background

The AYO™ WX Fit Testing Adaptor takes advantage of the Aimwell-patented Push 'n' Twist connection to provide an ergonomic and hassle-free fit-testing experience for respirators across the Aimwell AYO range.

The Adaptor comprises a plug-in module that fits between the Mask Assembly and Main Unit. The module includes a female Twist-Release socket and a male Push-Fit plug to seamlessly integrate into any AYO system.

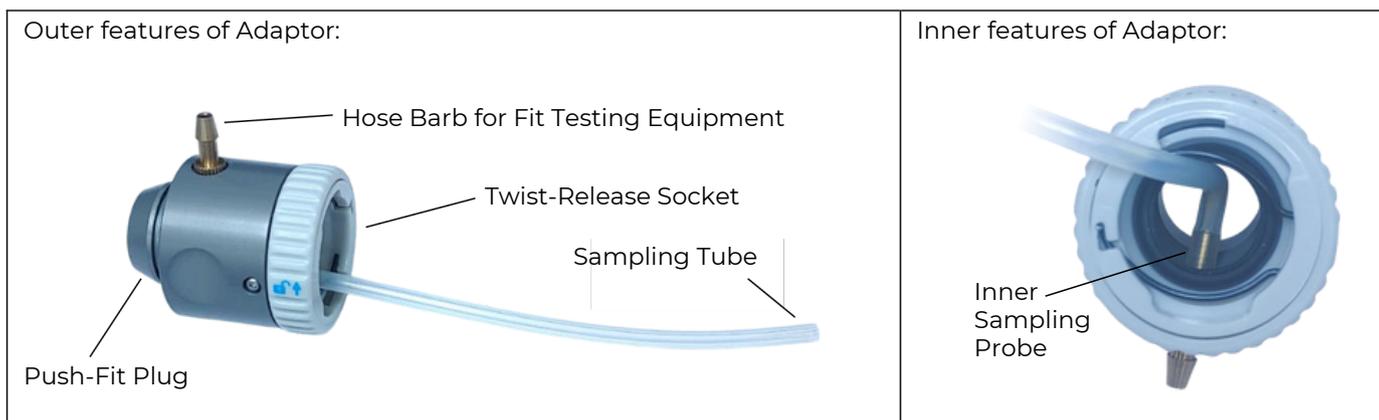

## Steps to Use:

 <p>1. Thread the sampling tube through the mask arm so that its end is positioned directly in the airstream of the user's exhalation.</p>	 <p>2. Push the Adaptor socket onto the mask plug until an audible click is heard indicating the latching springs are seated inside the groove.</p>	 <p>3. Connect the sampling hose from the Fit Testing Equipment to the brass hose barb.</p>
 <p>4. Don the Mask Assembly the same way as for regular use.</p>	 <p>5. Connect the Main Unit the same way as for regular use.</p> <p>Note the Power Unit now attaches to the Adaptor instead of the Mask.</p>	 <p>6. Conduct Fit Testing in accordance with your organisation's procedures.</p>